



In the best pharmacies and healthcare shops

## TOPICAL GEL FOR SCARS



[eshop.revee.it/en](https://eshop.revee.it/en)



Medical silicone  
Odourless  
Without parabens



CE

# REVÉE®

---

## SCAR GEL

**INGREDIENTS: 100% silicone gel** (cyclopentasiloxane, cyclohexasiloxane, dimethicone/vinyl dimethicone crosspolymer)

Silicone gels are the first treatment recommended by **international guidelines**  
**to cure and prevent** hypertrophic **scars** and keloids.

## COME FUNZIONA REVÉE® SCAR GEL?

### THYROIDECTOMY

scar

### TRAUMA

scar

### MASTOPLASTY

scar

### NEVUS REMOVAL

scar


### C-SECTION and ABDOMINOPLASTY

scar

#### FRESH SCAR

EPIDERMIS


DERMIS

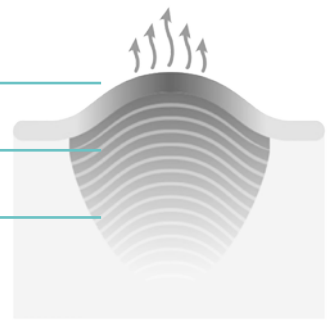

1. EXCESS WATER LOST through top layer of skin
2. INFLAMMATION retained within dermis
3. HIGH COLLAGEN PRODUCTION within deeper layers of skin

#### REVÉE® SCAR GEL

HUMECTANTS for additional moisturisation

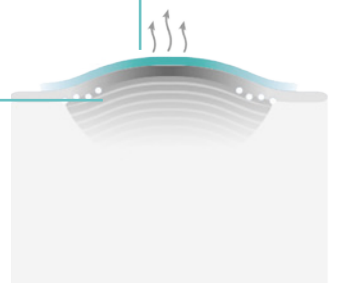

1. REDUCED WATER LOSS keeping scar tissue hydrated
2. LESS INFLAMMATION caused within dermis
3. LESS COLLAGEN PRODUCTION within deeper layers of skin



It reduces redness and itching, while improving the aesthetic aspect. The scar becomes smoother and more elastic.



Odourless.



Quick-drying silicone protective film.

Apply to the closed wound.

Let it dry for 5 minutes.

Repeat the treatment for 60-90 days.

Apply twice a day.



#### \*International guidelines

Hsu et al. 2017 "Review of Silicone Gel Sheeting and Silicone Gel for the Prevention of Hypertrophic Scars and Keloids." Wounds. 2017 May;29(5):154-158.

Mustoe TA et al. International Clinical Recommendations on scar management Plast Reconstructive Surg 2002; 110:560-71

Nast A et al. "German S2k guidelines for the therapy of pathological scars (hypertrophic scars and keloids)" JDDG 1610-0379/2012

Sukwha k et al. "Update on Scar Management: Guide Lines for Treating Asian Patients" Plastic and Reconstructive Surgery Dec. 2013, Vol 132 n.6 Scar Management in Asian Patient

#### REVÉE srl

C.so Quintino Sella 131  
10132 Turin, Italy

#### Customer Care

Mo / Fr 9:00 AM- 6:15 PM  
support@reevee.it

www.reevee.it  
@reeveeofficial

