

REVÉE®

Revée® Cardio

Compression Chest belt

DESCRIPTION,
WARNINGS
AND INSTRUCTION FOR USE



APPLICATION FIELDS

The **Revée Cardio** Chest Belt is a compression garment suitable for fixing the dressings, to be worn during the post-operative phase of cardiac surgery, sternotomy, thoracic surgery, gynecomastia or reconstruction of the pectoral region.

FEATURES

Breathable, comfortable and contoured, equipped with a flexible paravertebral splint, high-restraint elastic fabric, four adjustable velcro closures, and adjustable 2.5-cm shoulder straps.

MAIN FABRIC COMPOSITION:


87% polyamide, 13% elastane

CODE

REV.032

SIZE

Approximate Size Chart

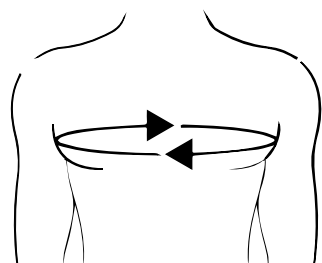

SIZE	BUST (CM)
S	78-88
M	89-98
L	98-108
XL	109-118
XXL	119-128

1. Bust

Measure the circumference at the highest point of the chest keeping the measuring tape horizontal.

HOW TO PUT ON REVÉE CHEST BELT

- make sure that the size label is placed inside the garment
- unfasten all the velcro closures and hooks in the front of the garment
- place the chest belt open on the bed
- lay down the patient on his back on the open garment, with his arms stretched forward if possible
- apply a slight traction and fix the four velcro fasteners on the velvet front part
- wear the shoulder straps, close and adjust them according to needs
- check that the degree of restraint obtained is satisfactory as well as comfortable for the patient

CONTRAINDICATIONS

Polytrauma with lung injury, do not put in contact with injured skin.

WARNINGS

For a longer life of the product, it is advisable to respect the following washing instructions. Hand-wash with lukewarm water, avoiding the use of fabric softeners. Wash the Revée® garments separately or together with fabrics of the same colour. Do not iron and do not dry clean. Preferably to be laid out horizontally. Dry thoroughly, away from sunlight and direct heat sources (such as radiator). High temperatures may compromise the elasticity of Revée® garments, and hence their therapeutic efficacy.



WARRANTY

Medical devices are guaranteed by the law.

For manufacturing faults, parts subject to wear and tear are excluded.

CLASS 1 MEDICAL DEVICE



REVÉE srl

C.so Quintino Sella 131, 10132 Torino

tel. (+39) 011 89 81 276 - P.IVA 10818580010

Made in Europa



	warnings
	CE marked product
	legal Manufacturer
	batch number
	see instruction for use
	recyclable product
	no sterile
	latex free material